


VGA Intra Oral Camera User Manual

Ver: ZZ-0058

VGA Intra Oral Camera User Manual

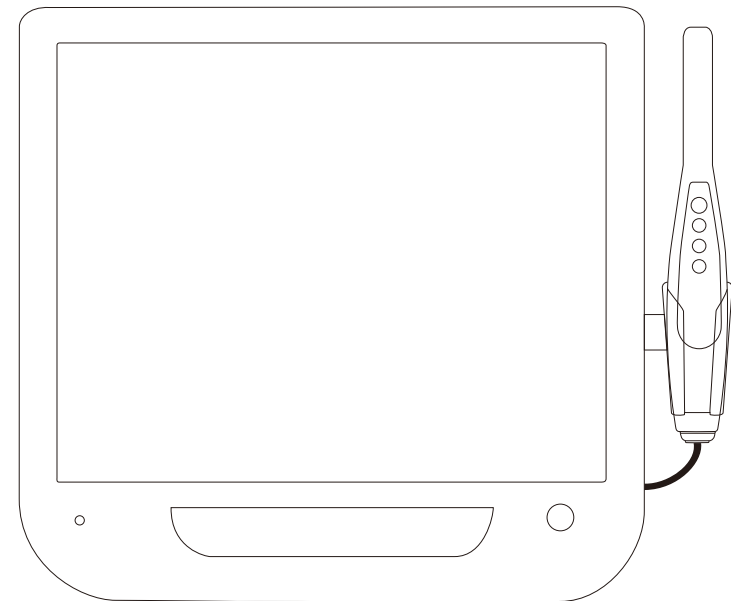

1. General Information

002 General Introduction

003 Specification

2. Assembly Instruction

006 VGA Monitor

006 VGA Camera

007 Intraoral Camera Holder

3. User Guide(008–010)

4. Other Information

011 Troubleshooting

012 Maintenance

013 Warranty Policy

General Information

Parameter

Image Sensor: CMOS1/4

Operating System: Embedded
Operating System

Picture Display: 1 Picture/4 Pictures

System Operation: Remote Control
Key Control

Backup Mode: Backup to SD Card
Automatically

Storage Capacity: Max 256G

Picture Storage Format: JPG

Focus Range: 5MM--50MM

Consumption: 30W

Specification

Standard Package


NO	Part Name	QTY
1	VGA Monitor	1 Pcs
2	VGA Camera	1 Pcs
3	Remote Control	1 Pcs
4	DC Adapter (12V)	1 Pcs
5	DC Cable	1 Pcs
6	Intraoral Camera Cable	1 Pcs
7	Intraoral Camera Holder	1 Pcs
8	SD Card(4G)	1 Pcs
9	Card Reader	1 Pcs

Option


NO	Part Name	QTY
1	Stand	1Set

Specification


VGA Monitor

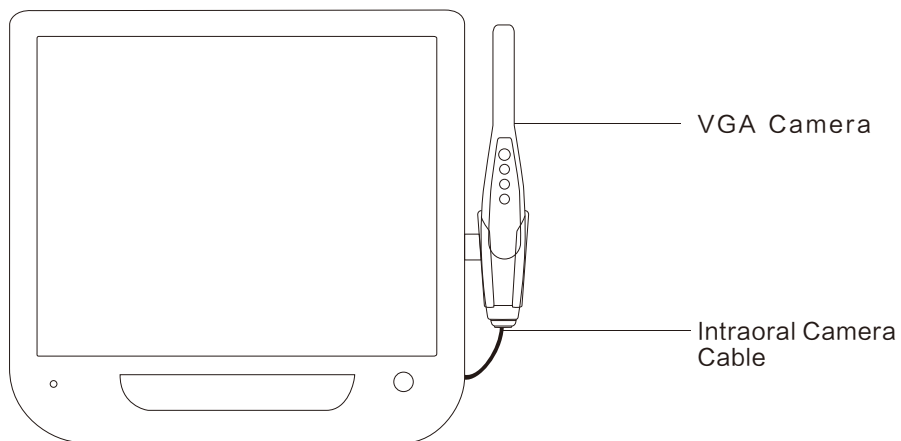

Remote Control

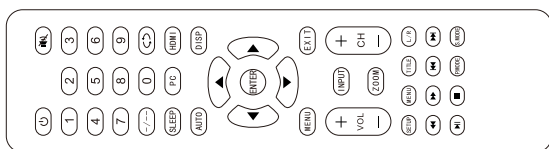

Specification


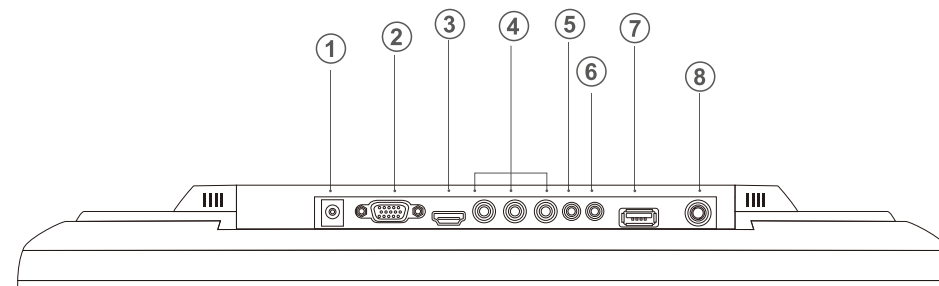


VGA Monitor

- ① DC Input 12V
- ② VGA
- ③ HDMI
- ④ AV
- ⑤ Audio Input
- ⑥ Audio Output
- ⑦ USB Input (For playing video, music, and picture)
- ⑧ Antenna or CCTV Input Interface

Assembly Instruction


VGA Monitor

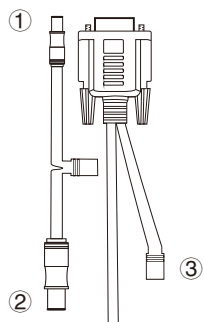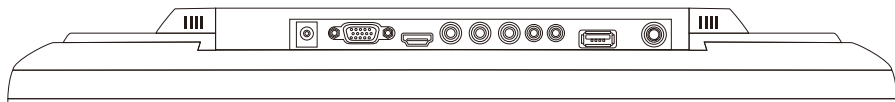


Remind: 1.Please connect port ① to the DC Input of monitor.
2.Please connect port ② and ③ for camera connection.

Assembly Instruction

Intraoral Camera Holder

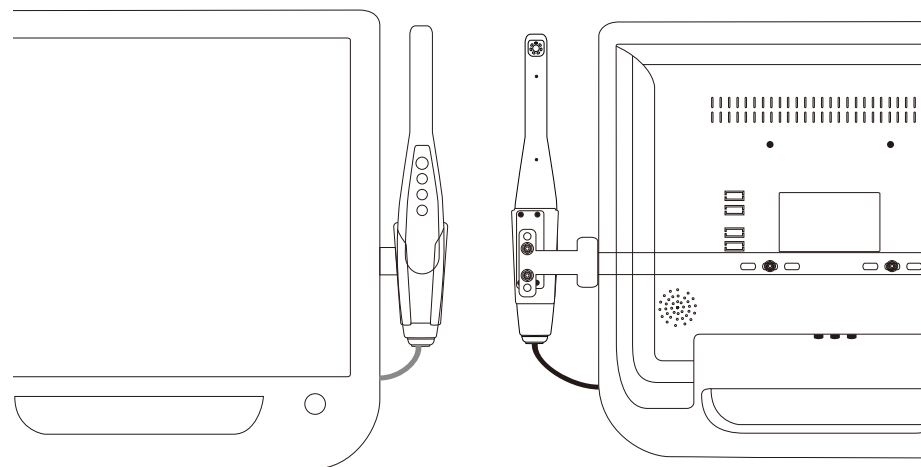


Bracket

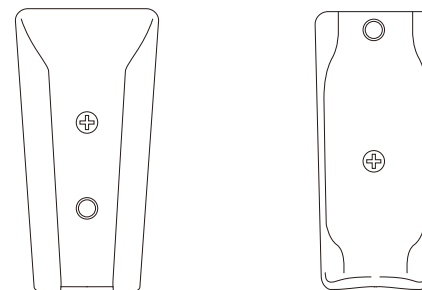

User Guide


Remote Control

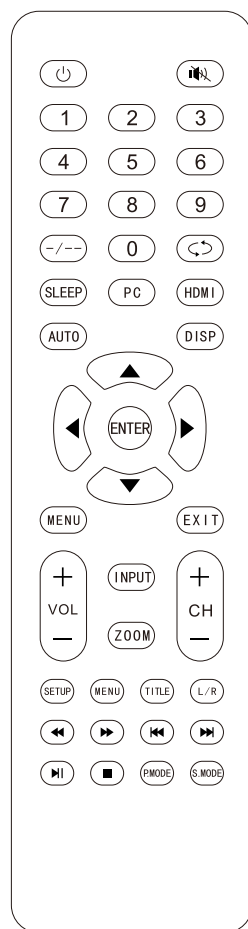


User Guide

Power On

Turn on the main power, then press the POWER button, the power indicator will become blue.

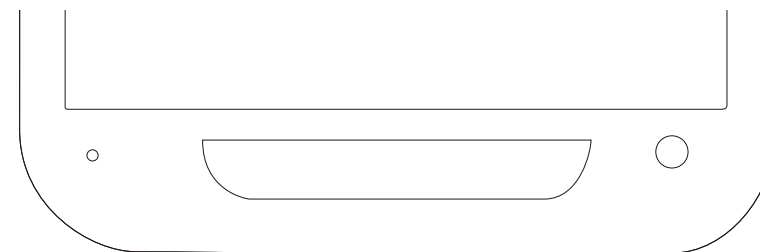


Input Signal Selection

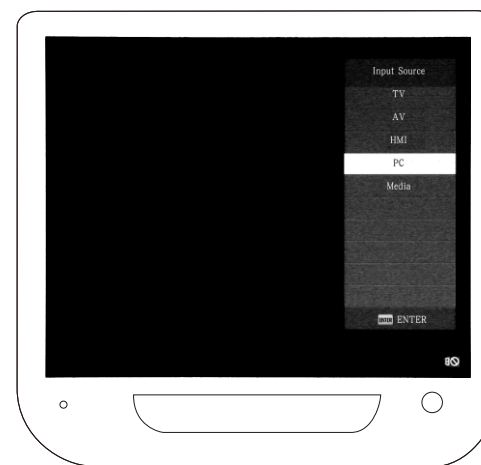

- After power on, press the ENTER button to show the signal list.
- Scroll ▼/▲ button to view the desired input source.
- Press ENTER or ► button to select the signal source.

- (1) When uses the Intraoral Camera System, please select COMPUTER mode.
- (2) When plays the video, please select MULTIMEDIA mode.

User Guide

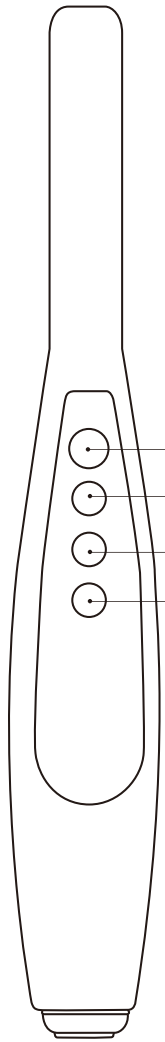

Key Function:

Key1: ①Capture ②Take Picture
③Turn off Camera(Long Press)
④Save Picture

Key2: ①Switch Picture Mode
(Single Picture ⇄ Multi-Picture)
②Cancel Picture
(Under Mode of Capture)
③Take Video(Long Press)

Key3: ①Play Forward
②Delete Current Picture or Video
(Long Press Under Mode of Browse)

Key4: ①Play Backward
②Delete All Pictures
(Long Press Under Mode of Browse)

Side key: Turn on / Turn off Camera

The camera should use with the camera sleeve.

Other Information

Troubleshooting

If you have any question about the Intraoral Camera System, first refer to this list. If none of these troubleshooting tips apply, please contact the maker for technical support.

Issue	Support and Explanation
Monitor cannot turn on	Please check the adapter work or not.
Picture Quality	Please check whether there is smear on the screen or camera lens. If none, please contact the agent.
SD Card can not be read	Please check whether the SD Card can be read on computer.
Black screen or no signal	Inverter board or circuit board is damaged.
Picture cannot show fully or not in a right position	Press AUTO key on remote control to adjust screen automatically.
No image while using Intraoral Camera	Please check whether it connected well.

Other Information

Maintenance

1. Please read the Manual before using this product.
2. Please do not open the Intraoral Camera System without permit.
3. To protect the circuit board or product, please keep the signal connector away from the sharp goods, metal or liquid.
4. Please keep the power supply cable or Intraoral Camera Cable away from stress.
5. Please take off the plug or turn off the power, if the Intraoral Camera System won't be used for a long time.
6. Please use the DC Adapter specified by the maker to avoid the damage from product. If the power supply cable or plug is broken, please stop using and contact the agent in time.
7. Keep in a dry, cool and well-ventilated place. Avoid exposure to excessive heat, light and air for prolonged of time. Keep away from strong acids, inflammables and explosives. Clean the surface of Intraoral Camera System regularly.

Warranty Policy

Dear Customer:

Thanks for using the product . TO obtain better warranty service, please read carefully the following clauses and keep this warranty card properly. You should provide this warranty card for each warranty claim.

Deal Name		Contact	
Deal Add		Tel	
Customer Name		Contact	
User Add		Tel	
Product Name		Model	
Purchase Date		Part No	
Maintenance Record	Date	Defect or Maintenance Record	

Remark:

1. Please keep this warranty card carefully, which will be used for warranty service.
2. ____Year(s) warranty from product purchased.
3. The following are excluded from this warranty or need to charge material or service fee.
 - 1) Warranty card can't be provided.
 - 2) Defect or damage caused by incorrect using.
 - 3) Artificial and abnormal external impact.
 - 4) Defect or damage caused by the man without maker' s authorization.
 - 5) Defect or damage caused by nature disasters or force majeure.
 - 6) The other artificial damage.

